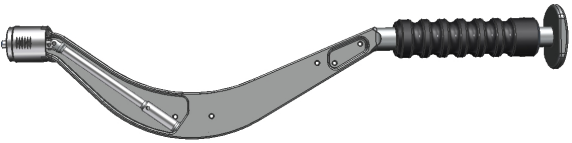

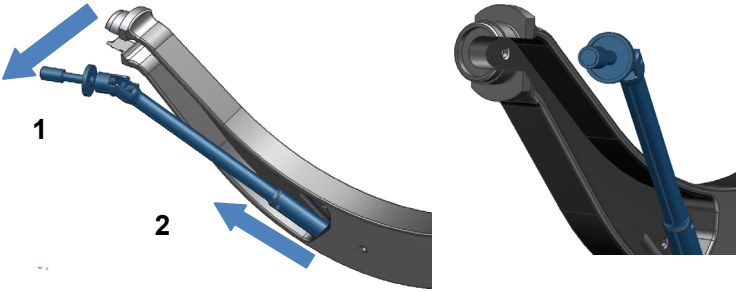
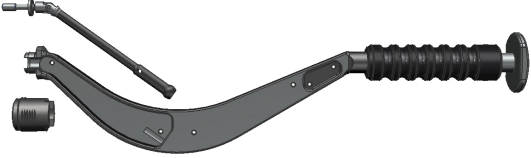


MIS-Seating Instrument Pyramid Cup Instruction for disassembly, cleaning and assembly

Start top for assembly

Read bottom-up for disassembly

<p>Fig. 1</p> 	<p>MIS shell impactor in usable and assembled state. The instrument is sterilized in this state.</p>
<p>Fig. 2</p> 	<p>Disassembling: Pull apart instrument and adapter.</p> <p>Assembling: Insert adapter until latching; the thread has to be visible. Consider correct rotation position of the adapter.</p>
<p>Fig. 3</p> 	<p>Disassembling: Pull apart the cardan shaft laterally first (1), then from the upper guidance (2).</p> <p>Assembling: Insert the cardan shaft first into the upper (2) and then into the lower guidance (1).</p>
<p>Fig. 4</p> 	<p>MIS shell impactor in disassembled state. The instrument is cleaned in this state.</p>

All instrument components are sterilized by steam sterilization. Procedures following valid standards should be used for the cleaning and sterilization of reusable instruments. Further information is attached to the product package insert and is available in the brochure "preparation of reusable instruments".