

MIS Shell Impactor

Instructions for disassembly and assembly

REF: 800097

Sequence of steps for disassembly: 

Sequence of steps for assembly: 



Fig. 1

MIS shell impactor in usable, assembled condition. The instrument is sterilized in this condition

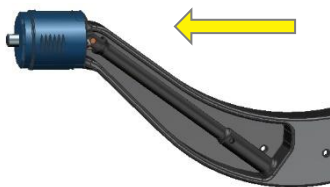


Fig. 2

Disassembling:
Pull off sleeve from instrument.

Assembling:
Slide the sleeve on until it clicks into place and the thread is visible. Make sure the sleeve rotates correctly.



Fig. 3

Disassembling:
Pull out the cardan shaft first from the side (1), then from the upper guidance (2).

Assembling:
Insert the cardan shaft into the upper guidance (2), then insert it into the lower guidance (1).



Fig. 4

MIS shell impactor in dismantled condition.

The instrument is to be cleaned in this condition.

All instruments must be sterilized with hot steam. When cleaning and sterilizing reusable instruments, validated procedures and systems that comply with applicable standards must be used. For further information, see the instruction for use of the associated implant and the brochure "Processing of Reusable Instruments" (Lit.No. 502.C006).

Note: The above statements are an excerpt from the valid surgical technique "Pyramid Hip Cup" (Lit. No.: 501.D001)